

CLARIONMASK REF	..CLG.../3VR/clg...
USE:	8 HOURS A DAY
FREQUENCY of WASHING:	1 WASHING EVERY 7 DAYS
MAXIMUM number WASHING:	30 TIMES
EFFECTIVENESS:	210 DAYS (1 wash every 7 days for 30 times)



TYPE II SURGICAL MASK

Intended to limit the transmission of infectious agents between people, patients, clinical staff and the surrounding environment.

ClarionMask performance:

- **Filtering efficiency (B.F.E.):** > 98%
- **Breathing resistance:** < 17 Pa/cm²

Five layers of fabric:

- 1° coloured outer layer: 100% cotton
- 2°/3°/4° intermediate filter layers: 3 superimposed layers of polypropylene fibrils
- 5° inner layer (in contact with the face): 100% cotton

TECHNOLOGY AND CHARACTERISTICS OF THE FABRIC USED

The fabric used to make the product is treated by **Livinguard AG, Bahnhofstrasse 12, 6300 Zug, Switzerland**, with the exclusive Livinguard Technology formulation, that mechanically inhibits the activity of bacteria and viruses and prevents their proliferation (eliminating them from the surface), ensuring a prolonged and effective protection from the risk of contamination.

Livinguard declares and certifies:

- **Ability to inactivate the virus:** 99,9%

Protected By Livinguard: www.livinguard.com

WARNINGS AND PRECAUTIONS

For personal use only. Do not use if the device is damaged. Do not use in case of hypersensitivity to the components.

REPLACEMENT

The product must be replaced after one year from the production date indicated on the package and the maximum washes have been made. Used product must be disposed of in compliance with the legislation on separate waste collection.

METHOD OF USE

THESE INSTRUCTIONS MUST BE CAREFULLY RESPECTED EVERY TIME YOU WEAR THE MASK

1. Open the bag and remove the mask.
2. Place the mask on one hand and bring it close to the face.
3. Pass the elastics behind each of the ears to comfortably adapt the mask to the face profile and use the adjusters to tighten the elastics.
(the upper part of each elastic - including between mask and adjuster - it must be shorter than the lower one, to guarantee a better adhesion).
4. Adjust the nose bridge bar to fit the top of the nose profile .
5. Adjust the lower part of the mask to completely cover the chin.
6. The product is for strictly personal use and, even after washing, it can not be used by people other than the first user.

It is advisable to leave the mask in a dry place away from heat sources, wash your hands after touching or washing the mask to avoid cross contamination.

USE, STORAGE AND DURATION OF THE PRODUCT

Before using the mask for the first time, make sure that the packaging is intact and sealed and that the product is complete and suitable for the application.

For subsequent uses, make sure that, previously, the product has been used according to the indications provided and stored in a healthy environment and away from very high heat sources, with a temperature between - 10 ° C and + 40 ° C and a relative humidity lower than 80%.

Maximum daily use time: 8 hours

Wash frequency: every 7 days, unless different specific needs

Maximum washes: 30 washes

Washing method: by hand in cold running water (max 30 °) without any type of detergent and without wringing, do not dry in direct sunlight. Do not use the dryer and do not iron.

To avoid compromising the filtering effectiveness of the product, do not tamper with, replace or damage any part of it.



CLARION LIVING SRL

Piazza Martiri della Libertà n. 4 - 28100 Novara (NO) - ITALIA

Tel: +39 0321 629249 - Fax: +39 0321 393720

Partita IVA e C.F.: 02631140031

Pec: clarionlivingsrl@pec.it - Email: info@clarion-living.com

Website: www.clarion-living.com

